



EASYTON



New Level of IOP measurement


*transpalpebral
digital iop
tonometer*

safe, quick and reliable

IOP CONTROL

We asked hundreds of eye care specialists – there is no 100% accurate technique of IOP measurement. We offer Easyton – a new method which provides results most close to the true intracameral IOP.

100
measurements
in 1 touch

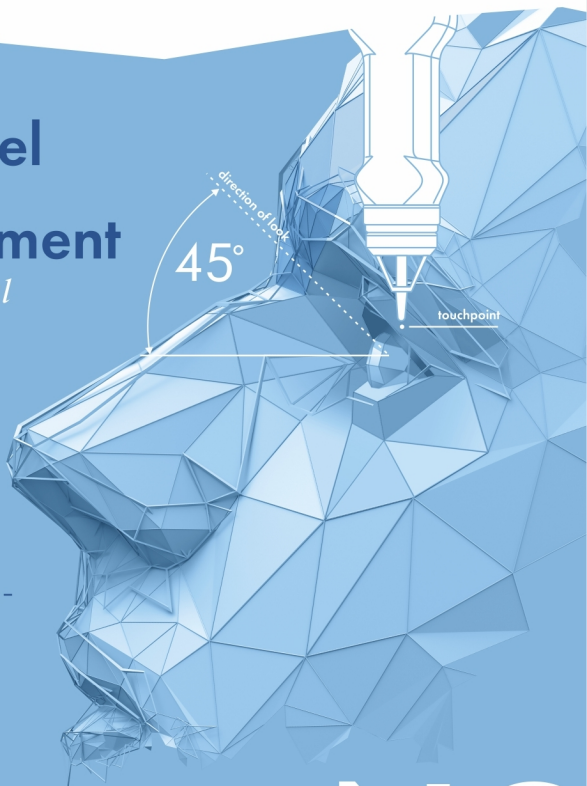

NO
corneal
touch



USE EASYTON FOR:

Ophthalmology
Optometry

Orthoptics
Fast screening any place

EASYTON *Experience the benefits*

Can be used by the most of eye care professionals

Can be used when the corneal tonometry is contraindicated

Safe to the cornea

Convenient for the doctor, gentle to patients

Cost-effective

Question 1. *Does the thickness of the eyelid influence the results of the IOP measurements?*

EASYTON's vibrating measuring unit and the calculation algorithm of EASYTON's software are designed to eliminate the thickness of the eyelid. There are 100 vibrations per 1 second, 80 of which serve to compensate the eyelid influence and 20 vibrations are done to get the most accurate IOP value.

Question 2. *Is accuracy of EASYTON IOP measurements proven?*

Accuracy of EASYTON measurements is proved with clinical trials on cadaver eyes. Additionally, there is comparison with Goldman tonometer which proves the validity of EASYTON measurement results.

Question 3. *What is disinfection and sterilization method for the EASYTON measuring unit?*

Sterilization is not required. EASYTON's measuring rod touches the skin of the eyelid only. Disinfection with an antiseptic wipe is enough.

**Accuracy
confirmed
with clinical
trial**



Get more information from manufacturer

Sales Department

16, Trubezhnaya Street,
Ryazan, 390000, Russia

Tel: +7 4912 293 418
global@elamed.com